

CAREVENT FIELD TEST

Team		Date:
Team Member:		
Demerit: 2 demerits for each omission, 1 demerit for wrong order		
For Failure to:		Demerit
1	Open case	
2	Inspect mask	
3	Connect the mask to the resuscitator	
4	Ensure both cylinders are full, minimum 200 bar (3000 psi)	
5	Ensure the oxygen cylinder connections are tight	
6	Open one cylinder, ensure gauge on the regulator is functioning	
7	Ensure the automatic cycle functions	
8	Close the cylinder	
9	Bleed to zero with the manual CPR button	
10	Repack and report results to captain	
	Total Demerits:	
Signature of Judge		